

## Assisi Portable™

The disposable battery-operated PEMF model weighs about two ounces and is available with a single 7.5 inch therapy applicator. The device is non-pharmacological and non-invasive and can be conveniently incorporated into bandaging or casting allowing maximum freedom of movement for the animal. The device can be used manually or in a programmed mode, as recommended by an animal healthcare professional.

### Features

- Effective, simple to use and affordable
- Approximately 90 treatments of 15 minute increments
- Lightweight, conformable applicators easy to position for therapy
- Portable battery-operated devices can be readily transportable from one location to another
- Treats directly through wraps, dressings, casts, etc.
- Virtually labor free, easy to administer
- No known side-effects
- Non-pharmacological, non-invasive



### For more information

On the internet

<http://www.assisivetrx.com>

By e-mail

[joe@mtavet.com](mailto:joe@mtavet.com)

By phone

866.646.8684

By fax

262.364.2270

By mail

**Assisi Animal Health  
135 Chestnut Ridge Road  
Montvale, NJ 07645**

***Bringing the best advances in human medicine  
to the “other” members of your family***

# Instructions for Use

The Assisi Portable™ is for use only on your pet as instructed by your veterinarian


## 1. Safety Warnings

- There are no user serviceable parts inside the Assisi Portable™.
- Not protected for ingress to water. Do not get the Assisi Portable™ wet.
- EQUIPMENT not suitable for use in the presence of a FLAMMABLE ANESTHETIC MIXTURE WITH AIR or WITH OXYGEN or NITROUS OXIDE or OXYGEN ENRICHED ATMOSPHERES.
- Insure that case and Applicator Loop are in good condition.
- Pulsed electromagnetic therapy SHOULD NOT BE USED on patients who have ANY implanted system that may contain a lead.
- Assisi Portable™ can interfere with appliances that are designed to detect and amplify weak radio signals. This includes television, radios, citizen band radios, potentially phones and other consumer devices in these categories. The interference may be heard as a repetitive tick in an FM radio and may cause tearing of a TV picture. These consumer devices are not generally well-shielded and are both producers of and subject to EMI from all sources.

## 2. Operation

Thank you for choosing Assisi, the state of the art in non-invasive, non-drug therapy for your pets. Please use as instructed by your veterinarian.

Press the power button (Ⓞ) to turn the unit on and off. Two green lights will blink on the front of the unit while the unit is working.

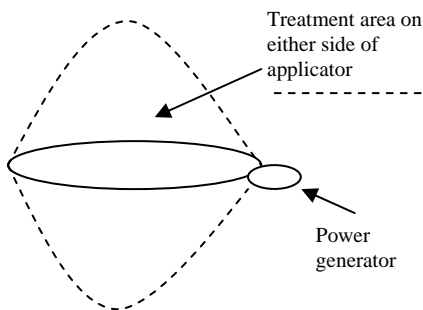

Put the applicator over area to be treated (within the dotted lines). You may have to hold it or tape it in place. Keep the applicator as round as possible.

If your veterinarian recommends treating for 15 minutes, press the power button to turn the unit on. Two green lights will flash during treatment. After 15 minutes, press the power button to stop treatment and turn off the lights and battery. Please use as instructed.

After treating your pet, if you do not turn the unit off while the lights are blinking, the unit will go into a sleep mode. In order to turn the unit off or return it to the manual mode, you have to wake the unit up. To do so, hold the power button down for 4 seconds, release it and the lights will start blinking, indicating the unit is providing a treatment. Then the unit can be used or turned off (by pressing the power button again). If the unit is allowed to remain in automatic mode, it will turn itself on and off at regular intervals reducing battery life.

If you don't know if the unit is off or in sleep mode, press the power button once. If the unit immediately begins to flash, then the unit was in manual mode. If the unit does not come on after just one press, then hold the power button for 4 seconds to wake the unit up. When the lights flash, you can treat your pet or turn the unit off by pressing the power button.

If your veterinarian prescribes automatic use, the unit is placed over the treatment area and left in place. It will treat your pet automatically for about 2 weeks.

Please visit [www.assisivetrx.com](http://www.assisivetrx.com) for further information. Thank you for choosing Assisi Animal Health